

DICOM Anonymizer Pro

Installation Manual

Table of Contents

1	ABBREVIATIONS	3
2	INTRODUCTION	4
3	SETUP PROCEDURE	4

1 Abbreviations

Abbreviation	Description
AE	Application Entity
DICOM	Digital Imaging and COmmunications in Medicine
DIMSE	DICOM Message Service Element
DLL	Dynamic Link Library
GUI	Graphical User Interface
LAN	Local Area Network
PC	Personal Computer
RAM	Random Access Memory
SCP	Service Class Provider
SCU	Service Class User
SOP	Service-Object Pair
TCP	Transfer Control Protocol
UID	Unique Identifier
USB	Universal Serial Bus
VR	Value Representation

2 Introduction


This document describes the steps that have to be performed in order to correctly install the NeoLogica DICOM Anonymizer Pro software on a target computer.

The DICOM Anonymizer Pro software can be installed and executed on a variety of different platforms, including Windows, Mac OS X, and Linux. A different *Installer* application is provided for each supported platform; please select and use the *installer* which is appropriate for your target platform. Here is a list of available *Installer* files and their association with each platform:


- **Windows:** *DICOMAnonymizerPro_win_32bit.exe* o *DICOMAnonymizerPro_win_64bit.exe*
- **Mac OS X:** *DICOMAnonymizerPro_macosx.dmg*
- **Linux:** *DICOMAnonymizerPro_linux_x86.sh* o *DICOMAnonymizerPro_linux_amd64.sh*
- **Other Java-enabled OS:** *DICOMAnonymizerPro_install_java.zip*

3 Setup procedure



In order to install the software on the target PC, please double-click on the installer file which is appropriate for your platform, as explained in the previous selection. The automatic *Installer* application will show up, allowing user to choose the language:

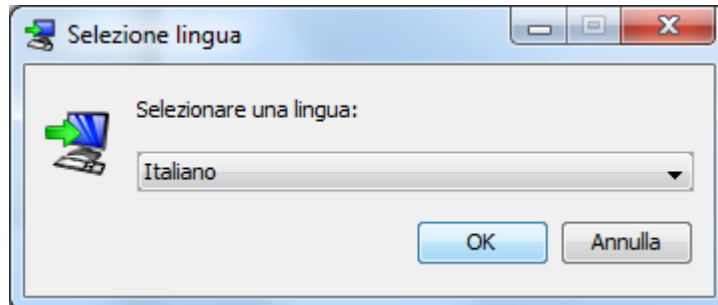


After choosing the language, the short installation procedure will start.

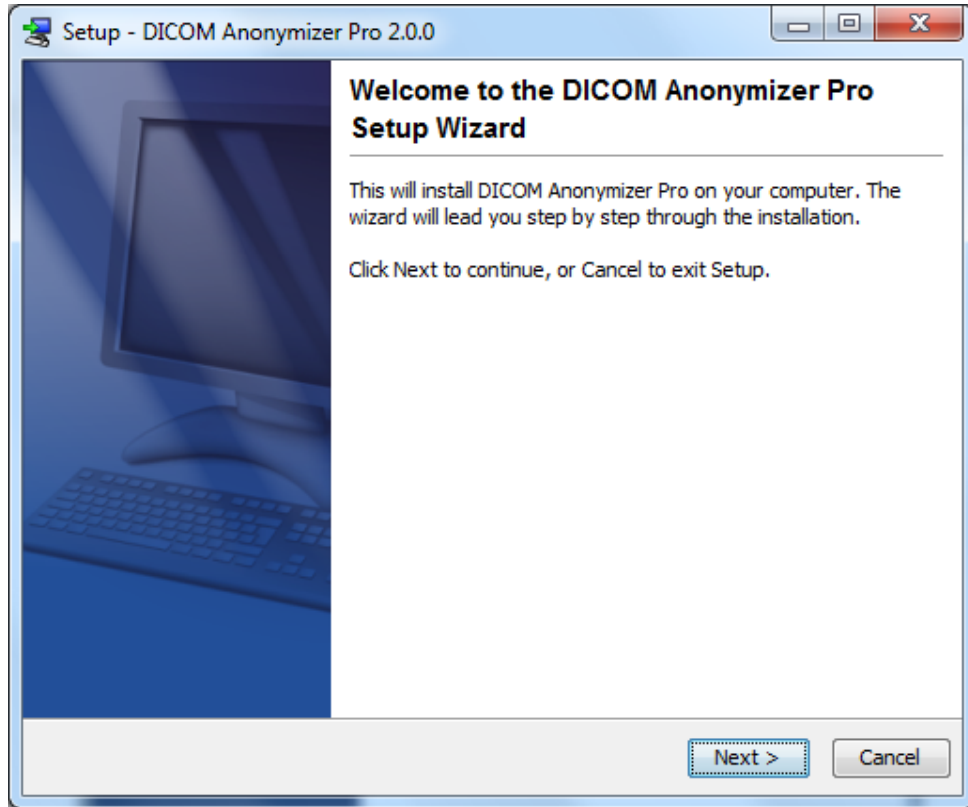

Clicking the *Next* button, you will be able to specifying the installation folder and the shortcuts folder:

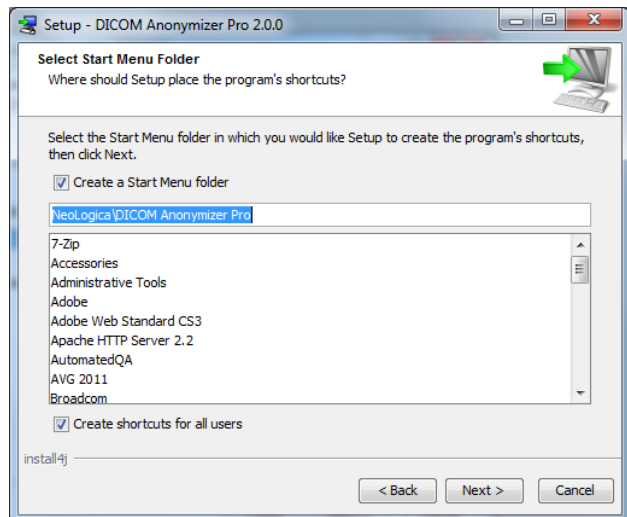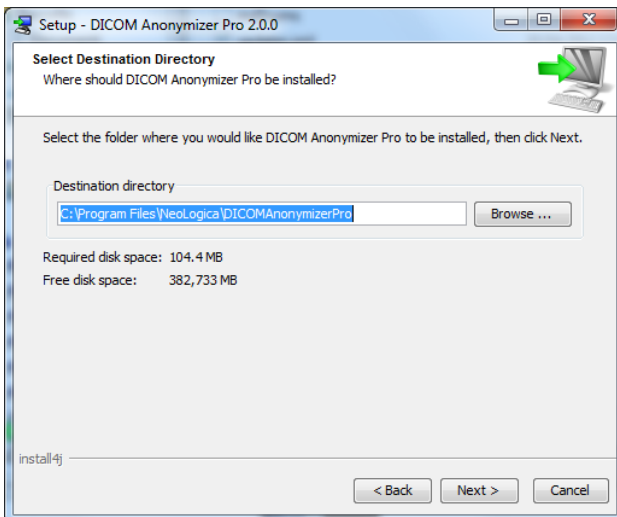

Press the *Next* button and wait for the finish of the installation process. Finally, the installation complete step will appear, confirming successful completion of the installation. Press the *Done* button in order to quit the *installer*.

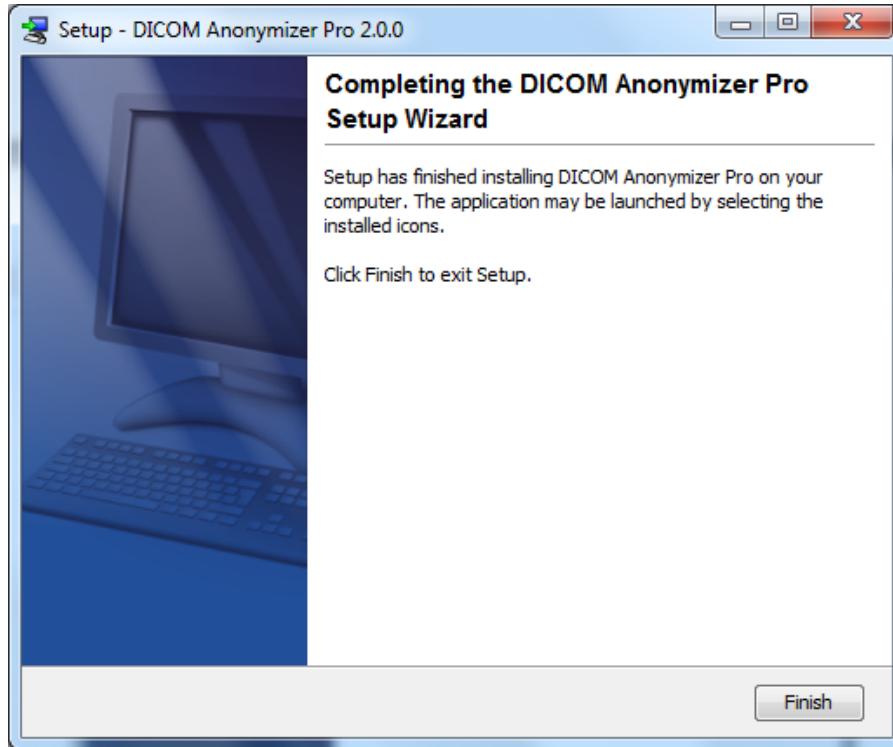

Now the DICOM Anonymizer Pro software is ready to be used. To learn how use it, please look at the *User Manual*.